

NEOPRENE STABILIZING KNEE BRACE WITH COMPOSITE HINGES

Model No: 37-107

Sizes: XSmall - XXLarge

Color: Beige

INDICATIONS:

- Osgood-Schlatter Disease
- ACL Sprains and Tears
- Patellar Instability
- Chondromalacia Patella
- Patellar Tendonitis
- LCL Sprains and Tears
- PCL Sprains and Tears
- MCL Sprains and Tears
- Medial/Lateral Meniscal Tears

BENEFITS:

This lightweight neoprene knee brace can be used for swollen or tender knees resulting from sprains, strains, sports injuries or post-operative rehabilitation. The lightweight composite hinges are strong and durable and can be used to give medial/lateral support and also prevent hyperextension of the knee through use of a stop in the hinge.

PRODUCT FEATURES:

- Lightweight neoprene for consistent compression and therapeutic warmth
- Inferior horseshoe support around knee joint for improved patellar tracking
- Lightweight composite hinges
- Adjustable support straps
- Easy slip-on style

HOW TO APPLY:

Insert foot into support and slide up over the knee. For a proper fit, the top of the support should be positioned approximately four inches above the center of the knee. Hinge axis should be aligned with the center of the knee and the inferior horseshoe should be directly under the patella. Secure with the adjustable hook and loop closures without limiting circulation.

CARE INSTRUCTIONS:

Close straps. Hand wash in warm water with a mild soap before wearing. Air dry away from heat. Do not tumble dry. Do not bleach.

SIZING:

Measure 4" above the center of the kneecap.

Size	Fits Knees
XSmall	12 - 13"
Small	14 - 15"
Medium	16 - 17"
Large	18 - 19"
XLarge	20 - 21"
XXLarge	22 - 23"

WARNING: Contact of this material with skin may cause irritation. Persons with a known susceptibility to dermatitis or with broken skin should avoid contact with this material. Anyone who develops skin irritation from contact with this material should discontinue its use immediately and consult a physician. Should not be worn by persons with phlebitis or circulatory disorders. Not recommended to be worn while sleeping.



800-327-4110



www.flaothopedics.com

©2002 FLA ORTHOPEDICS, INC.