

SAFE-T-SPORT® PATELLA SUPPORT

Model No: 37-300

Sizes: Small - XLarge
Color: Black

INDICATIONS:

- Chondromalacia Patella,
- Fat pad impingement
- Osgood-Schlatter Disease
- Patella instability
- Patella tendonitis
- Patellar subluxation
- Degeneration
- Arthritis
- Quad tendonitis



FEATURES & BENEFITS:

- **Sports Neoprene** provides therapeutic warmth and support around the joint to speed healing.
- **Adjustable Hook and Loop Straps** for an improved fit and enhanced compression control.
- **Simple Wrap-Around Design** for quick, easy application.
- **Inferior Patella Pad** applies uniform compression to reduce the load on the Patellar Tendon to increase stability, improve patella tracking, and help alleviate pain and pressure.

APPLICATION INSTRUCTIONS:

1. Open hook and loop closures on support.
2. Position the support behind the knee so that the strap ends are toward the front, and the support is centered on the crease behind the knee (Figure 1).
3. The lower left strap should fall just below the knee and lay in the natural indentation inferior to the patella. Pull the lower left strap over the knee. The inside tubular-shaped support pad provides compression to the patella tendon and should lay directly in the natural indentation. Pull the right lower strap to meet the left and feed the hook end through the buckle (Figure 2). Fold hook strap over to adjust compression and secure to loop patch.
4. Secure top straps with hook and loop. The neoprene should provide a comfortable amount of compression and should not feel constrictive. The kneecap should fit in the opening between the upper and lower straps (Figure 3).

Unfasten closures to remove support.

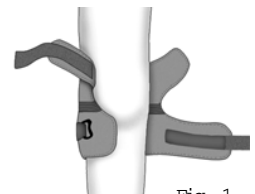


Fig. 1

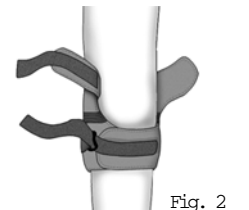


Fig. 2



Fig. 3

HOW TO SIZE

Measure circumference around the center of the patella (kneecap).

NOTE - If measurement falls between two sizes, select the next larger size.

Size	Fits
Small	13 - 14"
Medium	14 ½ - 15 ½"
Large	16 - 17"
X-Large	17 ½ - 18 ½"

CARE INSTRUCTIONS:

Secure hook and loop closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not use bleach.

WARNING: Contact of neoprene material with skin may cause irritation. Persons with a known susceptibility to dermatitis or with broken skin should avoid contact with this material. If skin irritation develops from contact with this material, discontinue use immediately and consult a physician. Should not be worn by those with phlebitis or circulatory disorders. Not recommended for use during sleep. NOTE: Consult your physician if you experience pain, swelling or unusual reactions.

800-327-4110



www.flaothopedics.com

©2007 FLA ORTHOPEDICS, INC.