

VENTILATED LUMBAR SUPPORT

Model No: 31-205

Sizes: Small - XX-Large

Color: White


SIZING:

Measure around hips just below the waistline.

Size	Fits
Small	28 - 32"
Medium	33 - 36"
Large	37 - 40"
X-Large	41 - 44"
XX-Large	45 - 48"

USE:

Stabilizes and supports the lumbar sacral and abdominal regions to provide relief from lower back pain and discomfort. Flexible stays provide strength to the lumbar region and an overlapping belt lifts and compresses the abdominal area for additional support. Use for occasional or chronic back pain, muscle strain, lumbar sprain/strain, or sacral contusion. Height is 10" at back and tapered in front.

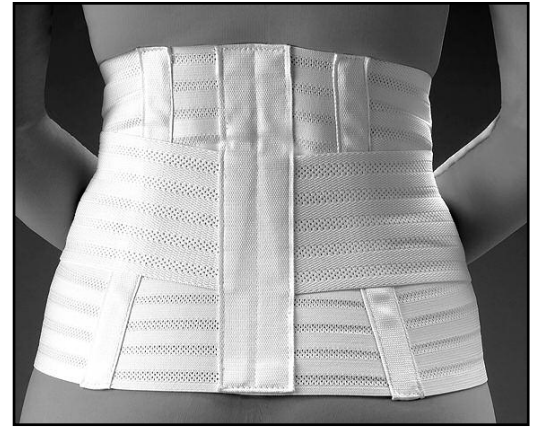

FEATURES & BENEFITS:

- High quality, latex free ventilated mesh elastic allows the skin to breathe for increased comfort and extended wear.
- Low profile for inconspicuous wear underneath clothing.
- Flexible stays provide strength and stability to the lumbar region.
- Overlapping abdominal belt provides additional lift and support in the abdomen area.
- Soft, plush lining is comfortable against the skin and non-irritating.
- Tapered panel contours the body for excellent compression and support.
- Easy application with simple hook and loop closures.
- Latex free.

WEARING INSTRUCTIONS:

Apply in standing position. Unfasten closures. Place around back and upper buttocks area. Pull side with hook and loop closure until the support holds the upper buttocks snug but does not bind the groin area when sitting. Close fastener on the support base. Pull the overlapping abdominal belt until snug and close hook and loop fastener.

CARE INSTRUCTIONS:

Fasten all closures. Hand wash in cold water with mild soap. Wrap in towel to remove excess water (do not twist or wring). Air dry away from heat. Do not machine wash or tumble dry. Do not bleach.



800-327-4110

Website: **www.flaorthopedics.com**

©2005 FLA ORTHOPEDICS, INC.