

# SOFT FORM® LUMBAR SACRAL SUPPORT 11" WITH CONTOURED STAYS

Model No: 31-560

Sizes: Small - 4XLarge  
Color: Beige

For Men & Women

Measure around the fullest part of the abdomen.	
Size	Fits
Small	28 - 32"
Medium	33 - 36"
Large	37 - 40"
X-Large	41 - 44"
XX-Large	45 - 48"
3X-Large	49 - 52"
4X-Large	53 - 56"

Packaged with Complete Care  
and Wearing Instructions.

800-327-4110

## INDICATIONS:

- Muscle Strain
- Lower Back Pain and Discomfort
- \* Mild Lumbar Instability
- Lumbar Spine Sprain/Strain
- Sacral Contusion
- Lumbar Disc Dysfunction

## BENEFITS:

Stabilizes and supports the lumbar sacral region to help provide relief from lower back pain and discomfort. Unique multi-layered soft laminate provides excellent support and comfort. Plush, latex-free elastic compression straps in a criss-cross design give compression to the entire lumbar region. Contoured rigid stays in the back provide firm and consistent stabilization.



## PRODUCT FEATURES:

- **Two Contoured Metal Back Stays** for firm and consistent stabilization. Stays are easily removed and shaped for a custom fit.
- **Two Flexible Side Stays** provide stabilization and keep the support in place.
- **Unique Sewn Five Piece Design** gives the ultimate tapered fit without binding or rolling.
- **Criss-Cross Plush, Latex-Free Elastic Straps** are designed to evenly distribute compression across the lumbar region.
- **Unique Laminate Material** combines three materials – durable outer in an attractive beige color, support foam base and soft cotton inner lining.
- **Easy Hook and Loop Closure** for easy application.
- **Two Anti-Roll Stays** are precisely placed to help keep the support in place and prevent rolling.
- **Latex Free**

## HOW TO APPLY:

While standing, wrap the support around the fullest part of your abdomen. Center on your hips with the back panel centered on your back. Once in place, fasten the closure on the support base with the elastic side straps loose. Next tighten and close the elastic side straps at the desired compression. The support should fit snug, but not too tight. The metal contoured stays can be removed and shaped by a qualified individual for a customized fit.

## CARE INSTRUCTIONS:

Fasten all closures. Hand wash in cool water with mild soap. Remove excess water and air dry away from heat. Wash as needed.