

PROLITE® LOW PROFILE 6" WRIST SPLINT

Model No: 22-470

Sizes: Small - XLarge

Color: Black

USE:

Provides stabilization and compression to weak or injured wrists. Ideal for sprains, strains, Carpal Tunnel Syndrome, and after cast removal. Removable palmar stay and two medial/lateral stays provide maximum support and stabilization, holding the wrist in a neutral position. Black color helps hide soiling. Ideal for latex sensitive people.



COMFORT BENEFITS:

- Made of an open cell foam with interlock polyester cover for a cool, breathable alternative to neoprene.
- Material provides comfortable, continuous compression.
- Low contour at the palmar crease allows for full finger dexterity and improved grip strength.
- Breathable spandex insert ensures a proper fit and easy application every time.
- Low profile and shorter 6" length for less bulk and comfortable all-day wear.



WEARING INSTRUCTIONS:

Unfasten closures. Slip wrist under the spandex insert and make certain that the thumb is inserted in opening. Position brace so palmar stay is centered on bottom of wrist. Place the two straps through the loop locks and fasten, starting with the lower strap. Re-adjust for a conforming fit. Cut hook straps to remove excess length if necessary.

CARE INSTRUCTIONS:

Fasten closures and remove stay. Hand wash in cold water and mild soap. Wrap in towel to remove excess water. Do not twist or wring. Air dry away from heat. Do not tumble dry. Do not bleach.

SIZING: Specify right or left. Measure around the center of wrist.

Size	Fits Wrist
Small	5 ½ - 6 ½"
Medium	6 ½ - 7 ½"
Large	7 ½ - 8 ½"
X-Large	8 ½" & up

800-327-4110



Website: www.flaorthopedics.com

©2004 FLA ORTHOPEDICS, INC.